

Pevco Link.2™

TOUCH SCREEN WITH
DELIVERY TRACKING



Pevco™

Healthcare's Pneumatic Tube Delivery System Expert™

pevco.com



Link.2 Features at a Glance

Send, receive, and track drugs, specimens and blood products easily and quickly.



Streamlined Maintenance

Option settings, network configurations and station diagnostics can be accessed by authorized staff, simplifying system support



Seamless Integration

Link.2 integrates with employee access system, providing users and managers with insight into every delivery



Document Deliveries

Securely track and document critical deliveries and know who sent and received high-value items



Monitor Transactions

See, in real time, the status of deliveries, empowering clinicians to stay focused on patient care



Built-in Help Screen

Reduce errors and simplify training with easily accessed answers to common user questions

Pevco Link.2™ provides a fast and intuitive way for clinicians to send and track items transported by pneumatic tube system.



Customizable Home Screen with Shortcut Access to Commonly Used Features

Hospital managers can configure the Link.2 home screen with up to 4 send types – Standard, Passcode, Tracked and Tracked Secure – empowering clinicians to send every item with the appropriate level of security.

And with short-cuts to features such as Settings, Carrier Tracking and Transaction History, Link.2 streamlines access to tools that ensure every delivery arrives on time.

Standard Send

Most common send type, simply select a destination from the short-cut page or the full directory

Passcode Send

For hospitals that require a more secure send type, recipient is required to enter a 3-digit passcode at the destination station to enable the carrier to drop into the station

Tracked Send

Using Pevco's built-in barcode scanner, senders are required to scan the barcode of a carrier, the contents and/or badge of the sender before sending to another station


Tracked Secure Send

Ensuring an extra layer of security, senders are required to scan the barcode of a carrier, the contents, and/or their badge and the receiver must scan their badge to retrieve a carrier




Empty Carrier Send

To make sure carriers are available where needed, Empty Carrier Send automates the redistribution of empty carriers to stations with low inventory

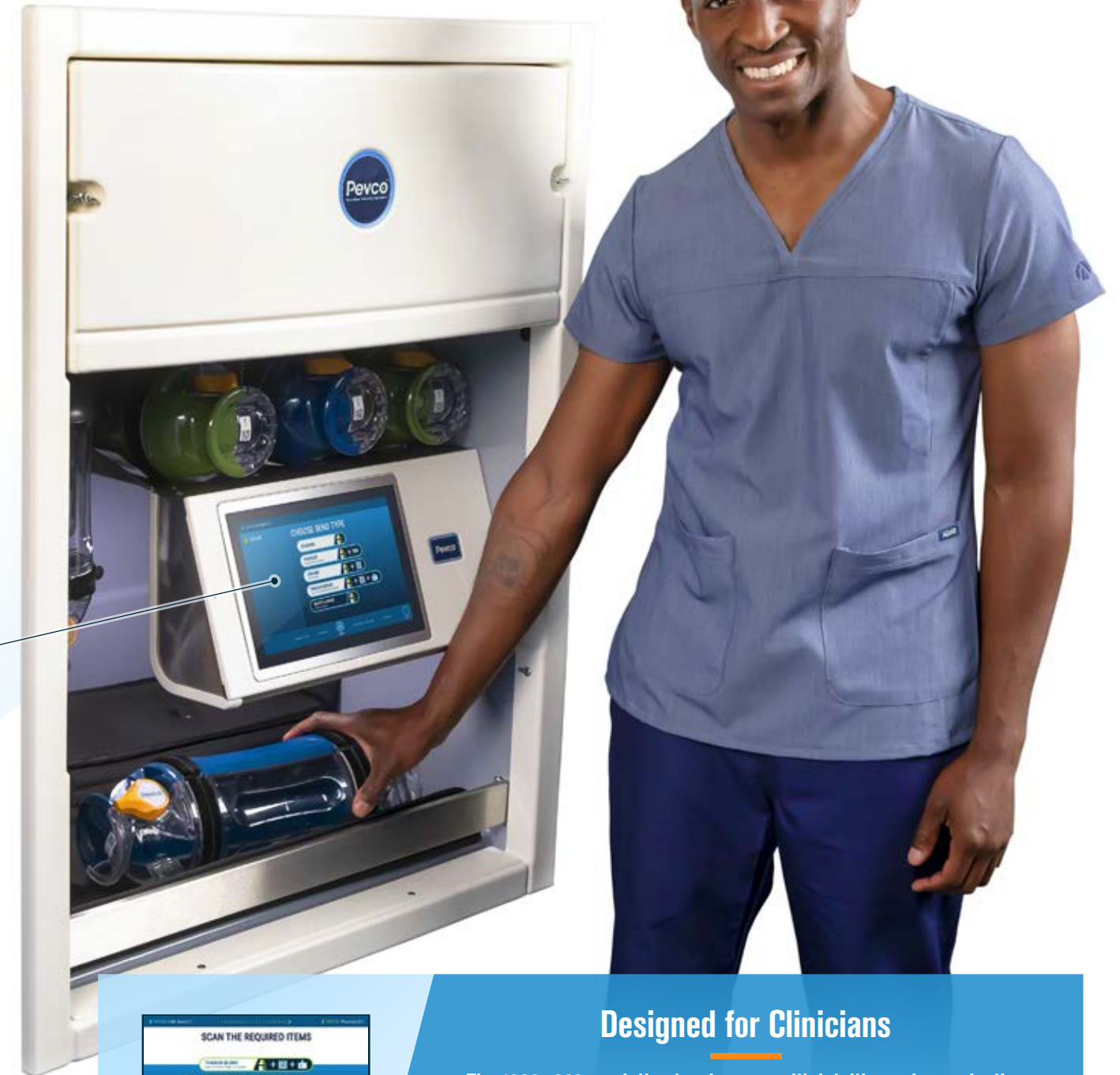

Designed for Clinicians

The 1280 x 800 resolution touchscreen with intuitive swipe navigation provides a clear sharp display – making the sending and tracking of drugs, specimens and blood products fast, safe and simple.

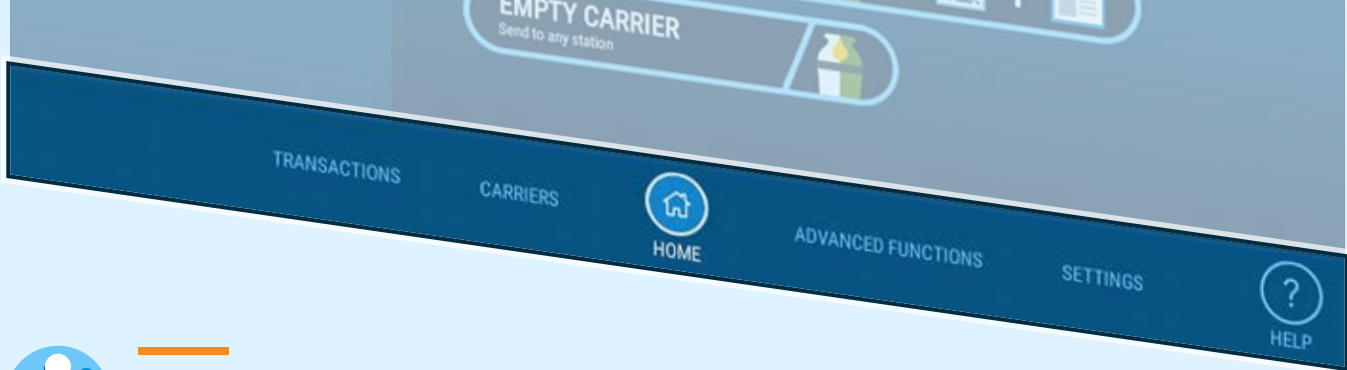

Advanced Functions Empower Users

Transactions

Review historical and pending status details of any transaction



Carrier Inventory

See details about a station's carriers at any time, including carriers allocated, carriers detected, and incoming carriers



Station Status / Home

Check relevant details when in Active / Non-active / Maintenance modes, choose send types and review station notifications

Advanced Functions

Authorized users can utilize functions that go beyond sending and receiving of transactions – diagnostics, emergency stop, maintenance lock and more



Settings

Technician staff can access and configure individual station settings, including network parameters and screen brightness



Built-in Help Screen

Reduce errors and simplify training with easily accessed help videos for basic send functions




pevco.com

1401 Tangier Drive
Baltimore, Maryland 21220

800.296.7382